



Bio-logic Navigator PRO
ENoG
ELECTRONEUROGRAPHY

Add the ENoG package to a new or existing Navigator Pro system to expand your patient services



The Bio-logic Navigator Pro AEP System with External SEP Stimulator option supports Electroneuronography

Assists in the diagnosis & treatment of facial nerve paralysis


- Evaluation of facial nerve viability is critical in management of facial nerve disorders
- Provides an objective measure of facial nerve integrity
- CPT Code for Electroneuronography — 92516

ENoG test protocol


- Electrical stimulation of facial nerve near mastoid with measurement of motor response recorded near naso-labial fold
- Compares response amplitude of the abnormal side to that of the normal side

External SEP stimulator


- Navigator Pro system triggers SEP stimulator to deliver stimuli & records responses
- Provides constant current pulses of brief duration for transcutaneous stimulation
- Output current is continuously variable over range of 1 to 100 milliamps

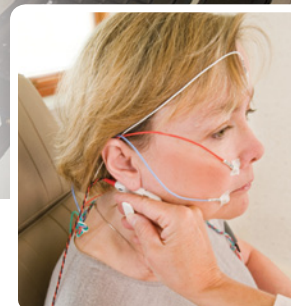

Bar electrode with handle is easy to place over the stimulation site

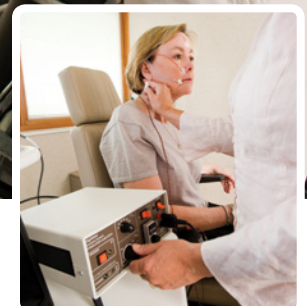

Digitimer DS7A SEP Stimulator

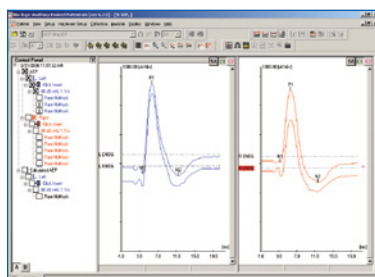

ENoG responses compared side-by-side on the Bio-logic Navigator Pro system

Your Bio-logic Representative:

