


Date	June, 2008
Label	Otometrics
Product	OTOSuite 3.0.0
PART number	8-49-75800
Affected items	
Reference	<i>ECO 24105</i>
Released by	Morten Høgholm Pedersen
Topic	Patch B1 for OTOSuite 3.0.0 fixes missing reflex screening data a. o.

Description

The following is included in "OTOSuite 3.0.0 - Patch B1":

- Acoustic reflex screening results are always "pass" in the infant report. Solution: Checkmarks are replaced by levels.
 - Cannot show the probe tone if only a single tympanometry curve is recorded. Solution: Fixed.
 - Incident #3966 "Reflex Screening not printed". Solution: All tags from Reflex Screening now available, and new report called Audiological Evaluation (Screening) created.
 - Probe tone is only shown for RT if it is different from 226 Hz. Solution: Fixed.
 - Probe tone is only shown for RT Left, so if only a right curve exists, the probe tone is not shown. Solution: New combined tag for RS and RD probe tone. Show probe tone for the ear that contains data. If probe tone for each ear is different, show nothing.
- Patch B1 also includes patch A (fixes failing fetch of audiogram from NOAH).

The distribution release number will end up showing "3.00.00 – Patch B1" after install:

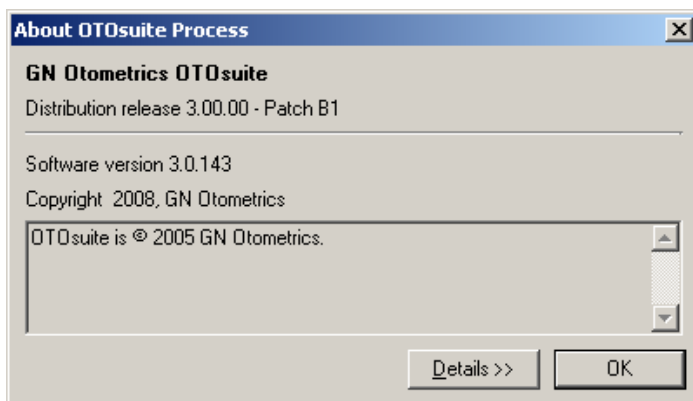

Installation

The patch will only install on OTOSuite build 3.0.143 (commercial release of 3.00.00) with or without patch A. Note that patch A is included in patch B1.

The patch is available on the Otometrics extranet www.otometrics.com under: Login > Software Updates > OTOSuite.

Unpack the zip file and run Patch_B1.vbs. Follow on-screen instructions.

Contact for more information

For more information, please contact your local Otometrics sales representative.