

New Software Version: AURICAL Plus v. 3.09
Instrument: Aurical
Program: PC Software

Released by: SVT Distributed by: JJM Date: 07-09-06

Part Numbers:

8-49-481/13 AURICAL PC 2-CHAN AUDIO V. 3.09
(released on 8-49-74300 AURICAL SOLUTIONS DISK, rev. 10)

Release date: 31-08-06

Replaces: Aurical Audiometer v. 3.08
(released on 8-49-74300 AURICAL SOLUTIONS DISK, rev. 09)

Firmware: It is recommended to use SPU software version 2.41 or later.

Software: When using Noah 3.x then please install the latest patch or version,
which can be downloaded from:
<http://www.himsa.com/frames/download.htm>

Update requirements: It is recommended to update older versions of the
software since this version contains critical bugfixes (see)

Description:

<i>Saving graphical speech data</i>	Binaural SRT speech curves (using phone and bone) are now saved correctly (including aided and unaided option).
<i>Saving Print Layout files</i>	The Print layout files (*.prl) is written into Print Layout section in aud020.ini when using "Save as Default" as in previous versions.
<i>Using AC transducers</i>	The program will always swap to the same AC transducers for both channels, when you toggle the output (phone/insert or insert/phone situations), i.e. you will then always get phone/phone or insert/insert output in 2 channel mode.
<i>Saving Switch box setting</i>	The switchbox settings are now saved correctly. The "Switch Box" button in COM Preferences is not shown when using MADSEN Conera.
<i>Installation</i>	The license is not removed when re-installing the program. The Installation program does not overwrite UserXX.ini (user setup) or *.prl (Print layout) files.
<i>Auto Threshold test</i>	A bug regarding the level field in Auto threshold test has been fixed. The level could be shown as a high negative number.
<i>The level in Graphical Speech mode</i>	The level in Graphical Speech mode can only go to 120 dB.
<i>Talk Over/Talk Back mode</i>	The Talk Over and Talk Back microphones can be turned on/off by pressing F12 to eliminate acoustical feedback in a special setup in the US. A parameter must be set in the aud020.ini file (located in the same folder as the program): [Configuration] WT TalkOver=1
<i>Noah certification</i>	This release is certified by HIMSA.
<i>Distribution media</i>	This version will be available on CD (part no: 8-49-74300).