

New Software Version: Aurical Plus REM 2.55
Instrument: Aurical Plus
Program: PC Software

Released by: SVT Distributed by: JJM Date: 28-07-06

Part Numbers:

8-49-121/21 AURICAL REM PC IMAGE
(released on 8-49-74300 AURICAL SOLUTIONS DISK, rev. 10)

Release date: 19-06-06

Replaces: Aurical REM v. 2.54
(released on 8-49-74300 AURICAL SOLUTIONS DISK, rev. 09)

Firmware: It is recommended to use SPU software version 2.21 or later.

Software: When using Noah 3.x then please install the latest patch or version,
which can be downloaded from:
<http://www.himsa.com/frames/download.htm>

Update requirements: It is recommended to update older versions of the
software.

Description:

NAL-NL1 setup

“Use Bone conduction values”-functionality has been fixed.

1. If you check “Use Bone...” and no Bone values are available then no target curve is shown. The target is shown if you uncheck the setting in this situation.
2. The effect of Bone values (when setting checked) should influence the target curves otherwise.

Printout

The print layout designer DLL (prnldxxx.dll) has been updated to the same version as for Aurical Plus Audiometer 3.05/3.06 to eliminate memory leaks for printout.

DSL i/o setup

The “Use measured UCL” text has been translated.

NOAH certification

The module is certified by HIMSA.

Distribution media

This version will be available on CD (part no: 8-49-74300).

Known limitations:

Installation

HIT.INI and DEFAULT.SET file gives overwrite warnings, when installing over old REM/HIT installations. You should always keep newer files and answer NO to overwrite.

Using Aventa 2.05

A problem regarding the COM port sharing is seen when using Aventa 2.05 fitting program and Aurical Plus Audiometer/Aurical REM. The Aventa development team at GN Resound is aware of this problem, and it will be fixed in the next release of Aventa.