

New Software Version: AURICAL Plus HIT v2.13
Instrument: AURICAL Plus
Program: PC Software

Released by: TK Distributed by: JJM Date: 23-11-2005

Part Numbers:

8-49-131/15 AURICAL HIT PC Software

Release date: 25-11-2005

Replaces: Aurical HIT v. 2.12

Update requirements: None

Description:

<i>Modulated noise</i>	Password for the Modulated noise feature is no longer required for neither old nor new Auricals.
<i>Online help</i>	Online help (PDF files) is now available in the following languages: English, French, German and Italian. The help can be invoked by clicking on the "Question" toolbar button or from the menu.
<i>Installation</i>	Installation problems under Windows XP/2000 solved. Optionally a shortcut in GN Otometrics group can be created to the HIT module.
<i>Background color</i>	Background color changed to blue.
<i>Noah module selection button</i>	NOAH module selection button changed to new Aurical plus look.
<i>Robustness</i>	Error handling greatly improved, error logging added.
<i>On-Top mode</i>	Bug fix: On Top mode now working correctly under NOAH 3 Save (Window) position function improved under NOAH 3. Resize improved, less flicker and toolbar is now always visible.