



RELEASE NOTE

New Software Version: Capella version 2.11 rev. 3
Instrument: Madsen Capella
Program: PC Software

Released by: KENESP Distributed by: Date: 27/3-2006

Part Numbers: Application software on CD ROM 8-49-74600 revision

Release date: March 27th - 2006

Replaces: Application software on CD ROM 8-49-74600

Also contains firmware 8-56-064 (version 2.12)

Update requirements: This is a maintenance update, update recommended but not required.

Install problem on Windows 2000

If you experience the error code 275 (Self-test error) please upgrade the firmware.

If the PC software fails on probe test please upgrade the PC software.

**New firmware
upgrade procedure:**

Insert the Capella solutions disk

Click on "Install Capella"

Under section 3 "Firmware installation" click on "Install"

**New PC software
upgrade :**

Insert the Capella solutions disk

Click on "Install Capella"

Under section 1 or 2 follow the installation guide in preferred language.

Description of corrections:

Correction of tolerances in ini-file

Capella PC software fails in probe test at customers due to error in present software build.

Extension of tolerances in Firmware power on self-test.

Self test error 275 (power selftest error) occurs on "GOOD" instruments

Window 2000 install fix

Software was not able to be installed on Windows 2000

NOAH 3 Certification performed

HIMSA NOAH 3 certification performed and passed